

Jib Transfer System – Sonar Installation Instructions


Begin with the system on the dock:



The only tool you will need is the allen wrench which is attached to the base


Installation steps:

1. Remove the bilge port cover from the deck of the Sonar

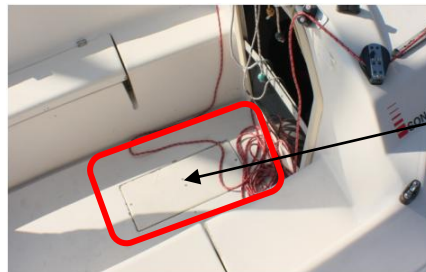

Remove the bilge port cover on the Sonar's deck

2. Using two people, lift the Jib Transfer System using the handles on the base and place the device in the Sonar – the seat needs to be positioned over the port side bench
3. Open the deck plate located in the center of the device's base

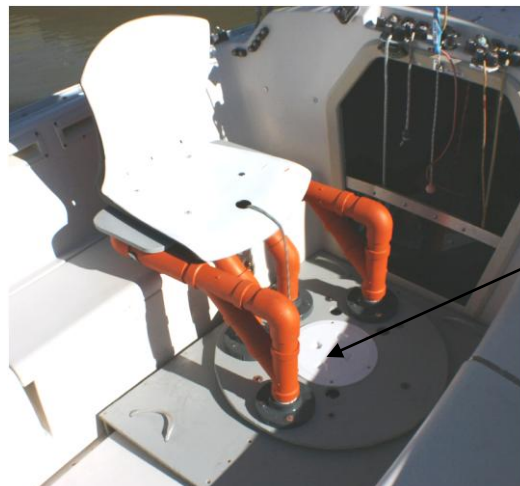

This is the deck plate that unscrews from the base

4. Remove the bolts from the securing mechanism and place them (with washers) through the four access holes in the upper base





5. Insert the first Aluminum support and secure it with the two thumb screws



This is what the device will look like from under the Sonar deck after step 4



5a. Insert the Aluminum support (vertically) into the deck plate opening

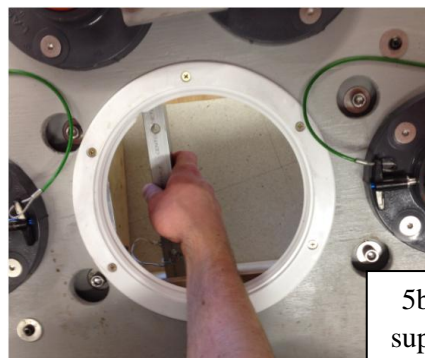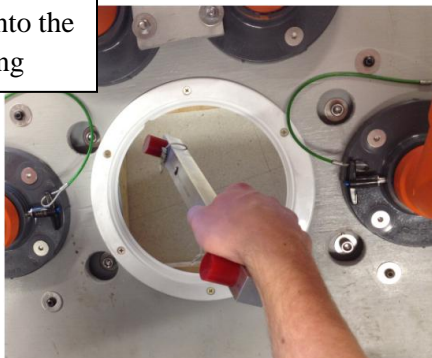

5b. Rotate the Aluminum support (horizontally) once it is under the deck

5c. Slide the Al bar up onto the bolts (making sure the red stoppers are contacting the underside of the deck)



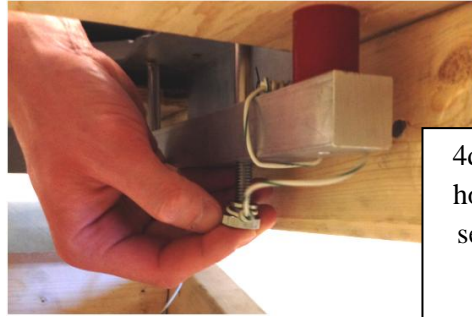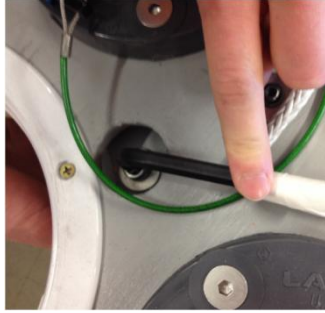

4d. Use the allen wrench to hold the bolt in place while securing the thumb screws from under the deck

6. Insert the second Aluminum support and secure it with the two thumb screws (see pictures above)

7. Replace the deck plate on the system base

The device is now installed and ready for use!